

## TECHNICAL DATA SHEET



**NAME:** CDIFF DISINFECTANT TABLETS

**PART NUMBER(S):** AB88

### PRODUCT DESCRIPTION:

**Cdiff Disinfectant Tablets** are the convenient and easy way to kill C. diff and many more organisms in just 4 minutes. These powerful tablets utilize sodium troclosene as the active ingredient and are bleach free. Sodium troclosene is much more stable than sodium hypochlorite (bleach), and releases chlorine at a much slower rate which means that C.diff Disinfectant Tablets produce a solution that maintains disinfecting power for 3 days, in a sealed container, and 24 hours in an open container vs. bleach solutions that become inactive after a day. With a pH of around 6, C.diff Disinfectant Tablets are the perfect disinfectant for virtually any surface, including floors, in hospitals, medical offices, dental offices, clinics, day care centers, nursing homes, veterinary clinics, and more.

### Features and Benefits:

- A broad-spectrum disinfectant, virucide and mildewcide
- Effective in both hot and cold water
- Disinfects and protects against odor
- 3 year shelf life
- Ships DOT non-hazardous

### Effective Against Bacteria, Viruses and Fungus:

- Clostridium difficile (C. diff)
- Carbapenem resistant Klebsiella Pneumoniae
- Klebsiella pneumoniae
- Staphylococcus epidermidis
- Enterococcus faecalis (Vancomycin Resistant)
- Staphylococcus aureus (GRSA & MRSA)
- Poliovirus type 1
- HIV-1
- Salmonella enterica
- Trichophyton mentagrophytes
- Acinetobacter baumannii
- Herpes Simplex Virus Type 1
- Escherichia coli O157:H7
- Norovirus
- Canine parvovirus
- Newcastle Disease Virus
- Respiratory syncytial virus
- Pseudorabies virus

### Properties:

Appearance.....White tablet  
Specific gravity.....~ 1.0 (in water)  
pH.....5.5-6.5

### Directions

Complete directions on product label  
**To Disinfect:** Add 2 – 6.55 gram tablets to 1 gallon of water. Contact time: 10 minutes.  
For C difficile spores: Add 2 – 6.55 gram tablets to 1 quart of water or 8 – 6.55 gram tablets to 1 gallon of water. Contact time: 4 minutes.

### Authorizations

EPA Registered Product

### Quantities:

240 UNITS/CASE



[WWW.GREENCHEMLABS.COM](http://WWW.GREENCHEMLABS.COM)

**CUSTOMER SERVICE: 1-800-946-3151 | FAX: 1-800-336-9068**

